


## PAC-Guard Protective Visor

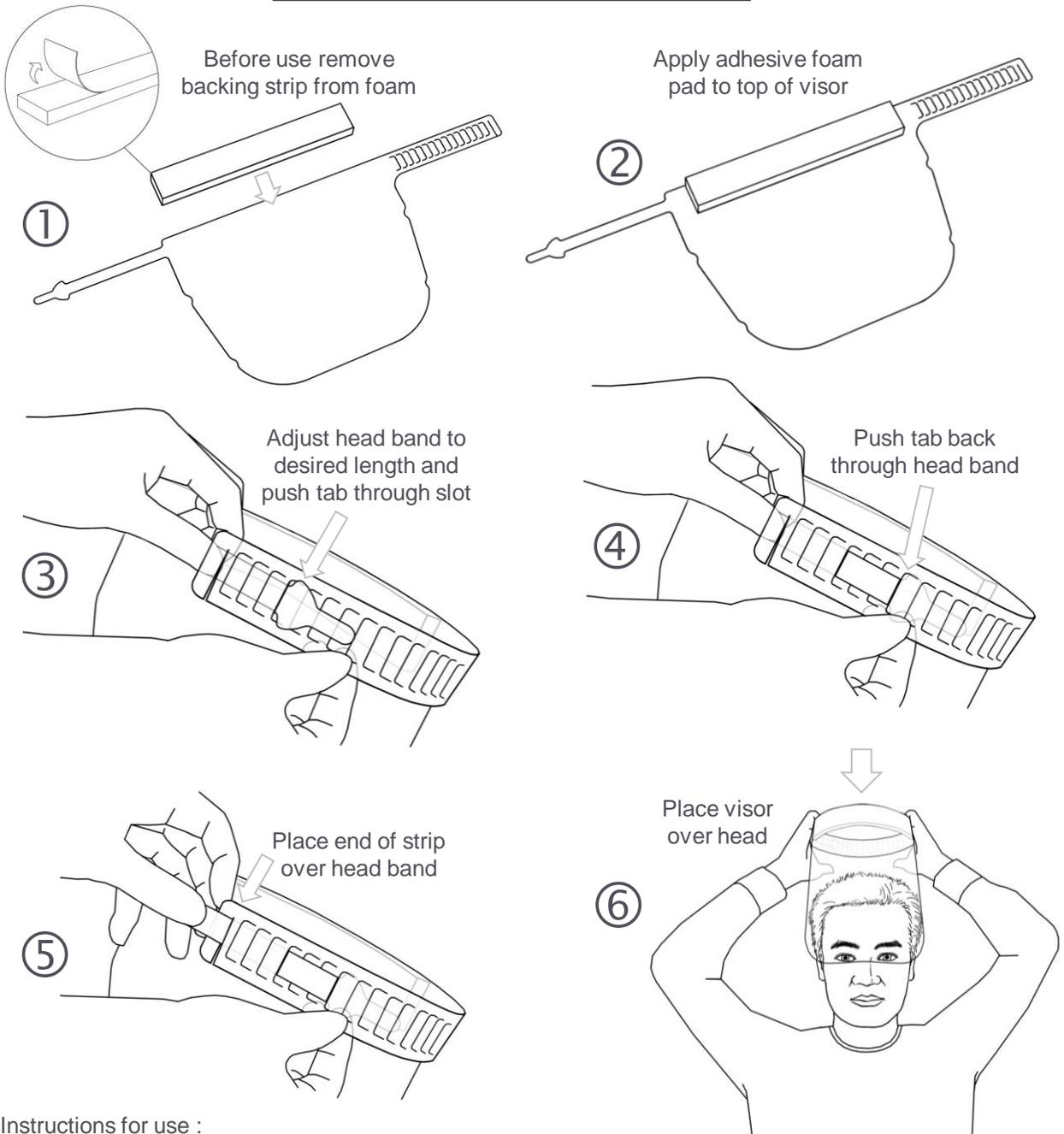

### Instructions for use :

*This is a general-purpose device intended to protect the face from nose and mouth splatters and airborne particles. This protective visor shall not be used for other purposes or as personal protection against other hazards.*

*Wash your hands before and after using this equipment. Remove when eating, drinking and smoking. Do not expose to temperatures above 55°C or below 0°C.*

*Clean with soap and water or hydroalcoholic gel, use a non-abrasive sponge. Dry and clean the surface with a lint-free cloth.*

*Replace the visor if the screen is scratched, blurred or damaged in any way. Do not use after maximum one year of purchase. Do not use if outer packaging is damaged in any way.*

*Latex-free foam, approved for skin contact. PET visor coated with anti-fog to help provide clear vision. Can be worn over eyewear or a face mask.*

*Compliant with Regulation (EU) 2016/425 (Category I) – the certificate of conformity can be consulted at [www.paccor.com/solutions/pac-guard-visor/](http://www.paccor.com/solutions/pac-guard-visor/)*

*Manufactured by: PACCOR UK Limited - Drum One - Drum Park - Chester le Street - County Durham - DH2 1AE – UK*